

ASSEMBLY INSTRUCTION for 7" Device on 110cm Desktop Pedestal

Thank you for purchasing the CAN-AM Howen AI Automated Temperature Screening kiosk.

For any support question, please call 1-866-976-4177 ext 1001 or email support@canamwireless.com

Instruction to setup the device with desktop pedestal:

1. Open the three bottom feet of the floor pedestal base



Figure 1

2. Identify the 3 black wire (1 power cable connector, 1 network RJ45 connector, and 1 USB connector) route it thru the center hole and the lock washer. Refer to figure 2.



Figure 2

3. Install and tighten the lock washer to bottom of the device pedestal. Refer to figure 3.



Figure 1

4. Install the bottom cover and the three bottom feet from step 1.