

## ASSEMBLY INSTRUCTION for 7" Device on 110cm Floor Pedestal #C

Thank you for purchasing the CAN-AM Howen AI Automated Temperature Screening kiosk.

For any support question, please call 1-866-976-4177 ext 1001 or email [support@canamwireless.com](mailto:support@canamwireless.com)

### Instruction to setup the device with 110cm floor pedestal:

1. Mount the tower using the provided 3 mounting screws at the bottom of floor pedestal tower. The screws should be in a poly bag, in the box that contains the base of the floor pedestal. Refer to figure 1.



Figure 1.

2. Open the top cover of floor pedestal tower by unscrewing the screws. Refer to Figure 2.



Figure 2

3. Remove the locknut at the bottom of the pedestal tube. Refer to Figure 3.

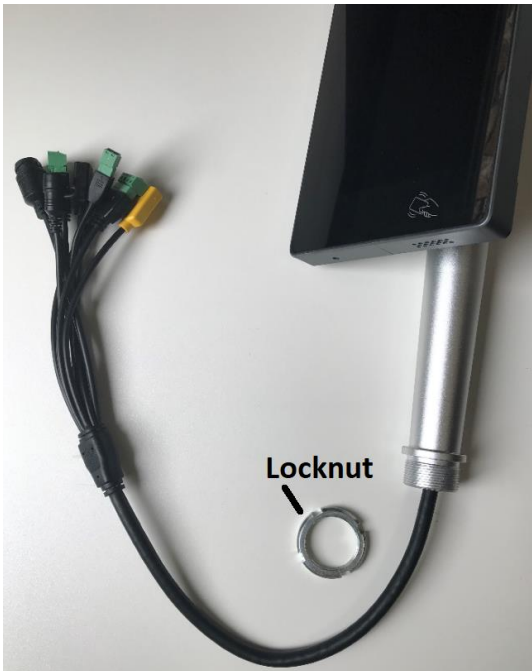


Figure 3

4. Route the black wires through the floor pedestal top cover and the locknut. Refer to Figure 4.



*Figure 4*

5. Secure the pedestal tube to the top cover by installing and tightening the locknut at the bottom of top cover. Refer to Figure 5.



*Figure 5*

6. Connect the following wires:
  - a. Power cable
  - b. Ethernet wire (RJ45)



*Figure 10*

7. Install the top cover to the floor pedestal using the 4 screws from step 1 above (screws are in a poly bag, in the packaging box that contain the base of the floor pedestal)
8. Connect the power adapter cable (and Ethernet cable if needed) to the base bottom of the tower.
9. Place the stand on a flat surface.